

I N S T R U C T I O N S



Adjustable Speech Mirror

Catalog Number 71800-0000

1. After unpacking your speech mirror, remove the film that covers the shatterproof plastic reflecting surface. The film is there to protect the surface from scratches during manufacturing.



2. To mount the mirror to a table, squeeze the clamp handles to open the clamp and grip the table top with the clamp jaws. The spring holds the clamp firmly in place.
3. The clamp may also be mounted on a round tube such as a bed rail, or wheelchair frame. The tube should rest in the V shaped notches in the clamp grip surfaces.
4. To adjust the mirror to any angle, hold the flexible gooseneck and bend it up or down, right or left until the angle is satisfactory.
5. Do not clean the reflective surface with abrasive cleansers. Use only soft cloths and water or a spray glass cleaner.

Maddak Inc.

6 INDUSTRIAL ROAD • PEQUANNOCK, NJ 07440-1993 USA
973-628-7600 • fax 973-305-0841 • email custservice@maddak.com • www.maddak.com