

I N S T R U C T I O N S



Children's Chaise™ Child Seat

Catalog Numbers 72706-1000 - Turquoise • 72706-1001 - Lavender
72706-1002 - Replacement Belt Kit

The included seat belt kit consists of the following:

- Two 18" belts with woven hook tape.
- Two 18" belts with woven loop tape.
- One 54" belt with both hook and loop tapes.

When using the chaise, be sure to use the seat belts as follows:

- 1) Before placing child in the chaise, attach the lap belt (the long belt) to the chaise by inserting its two ends through the slots from the bottom side of the seat and pull taut from the top side of the chaise, making sure that the hook-and-loop tapes are facing up. Place child in the chaise with thighs positioned on each side of the belts that are protruding through the belt slots. Loop each end of the belt over the thighs and through the belt loop attachments that are fixed to the two sides of the chaise as shown in **Figure 1 (A)**.
- 2) Attach a set of the shorter belts (one with hook tape, one with loop tape) at the waist level (bottom slots) by looping the belts through the slots as shown in **Figure 1 (B)**.
- 3) Attach the remaining set of the shorter belts at chest level by selecting one of the two upper sets of slots as shown in **Figure 1 (C)**. To determine proper level, make sure that the belts are always below the shoulders' level and under the armpits so that **THE CHILDREN'S ARMS ARE ALWAYS OVER THE BELTS**.
- 4) Head rest can be adjusted four ways to best suit the child.

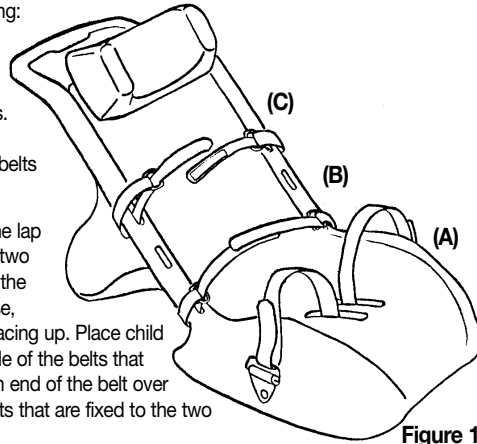


Figure 1

WARNINGS

- Never leave child alone in the chaise without adult supervision.
- Be sure to have the child securely fastened with seat belts when in the chaise.
- Belts pose potential choking hazards and, if not in use, should be removed from the chaise and kept out of reach by the child or by other children in the area.
- Never lift the chaise while the child is sitting in it.
- Never use in a moving vehicle.
- Never use as a floatation device.

Maddak Inc.

6 INDUSTRIAL ROAD • PEQUANNOCK, NJ 07440-1993 USA

973-628-7600 • fax 973-305-0841 • email custservice@maddak.com • www.maddak.com

972706101

0602

I N S T R U C T I O N S



Cleaning Procedures

Internal Cleaning:

To drain residual bath water from inside the chaise after bathing, tilt the chaise forward until the back rest is nearly vertical to allow bath water to drain out from the drainage holes.

To rinse the interior of the chaise, turn it upside down and pour clean water through the drain holes, then return chaise to the draining position to drain.

To prevent mildew from forming, occasionally spray interior through drain holes with commercially available spray disinfectant. Always rinse with water after using spray disinfectants.

External Cleaning:

The exterior of the chaise and the head rest can be cleaned by using any commercially available spray cleaner or mild soapy water; then rinse thoroughly with warm water.

The seat belts can be machine washed but should only be air-dried. **DO NOT DRY SEAT BELTS IN AUTOMATIC CLOTHES DRIER.**

Maddak Inc.

6 INDUSTRIAL ROAD • PEQUANNOCK, NJ 07440-1993 USA

973-628-7600 • fax 973-305-0841 • email custservice@maddak.com • www.maddak.com

972706101

0602