

# I N S T R U C T I O N S



## Shower Stool

72715-2000 Non-Rotating Shower Stool



72715-2100 Rotating Shower Stool

### Parts List:

1. Seat - 1 each
2. Leg - 4 each
3. Turntable – 1 each (Rotating model only)
4. Slip disk – 1 each (Rotating model only)

### To Assemble:

1. Place seat upside down on a flat surface
2. Insert legs into the recesses of the seat (see figure 1) by applying a firm downward pressure until the leg is fully engaged
3. Upright seat
4. Place slip disk on top of seat; aligning the center holes (rotating version only - see figure 2)
5. Place turntable over seat and insert the center stem through both center holes of the slip disk and seat (rotating version only - see figure 2)

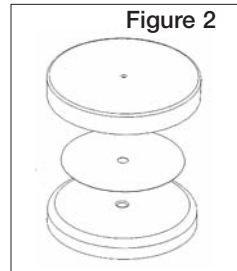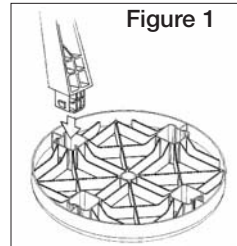

### Adjusting the Legs:

1. Rotate the leg extension and disengage from the leg
2. Insert the leg extension into the desired slot until fully engaged

### Care and Maintenance:

1. Clean with a non-abrasive cleaner
2. Periodically inspect all components for damage or looseness

### Warning:

- Do not stand on the shower stool for any reason
- Do not use this product without first reading and understanding the instruction sheet
- Weight capacity - 300 lb. (136.36 kg)

# Maddak

6 INDUSTRIAL ROAD • PEQUANNOCK, NJ 07440-1993 USA

973-628-7600 • fax 973-305-0841 • email [custservice@maddak.com](mailto:custservice@maddak.com) • [www.maddak.com](http://www.maddak.com)