

I N S T R U C T I O N S



Hole-in-One Button Aid

Catalog Number 73886-0002
U.S. Patent #5,737,951



1. Grasp the handle either by placing your hand through the hollowed center or around the outside.
2. Choose which loop best fits your button.
3. Slip the wire loop into the button hole and place the wide part of the loop around the button.
4. Pull with the handle so the narrow part of the wire is under the button.
5. Continue to pull until the button is pulled through the button hole.
6. When the button is in place, push the wire loop so the button is in the wide part of the loop and slip the button aid off.

When not in use the button loops fold back toward the inner curve of the handle for compact storage.

This utensil can be tightened at anytime by using a standard screwdriver.

Maddak Inc.

6 INDUSTRIAL ROAD • PEQUANNOCK, NJ 07440-1993 USA
973-628-7600 • fax 973-305-0841 • email custservice@maddak.com • www.maddak.com