

I N S T R U C T I O N S



Flo-Trol™ Cup Convalescent Vacuum Feeding Cup

Catalog Number 74588-0000



Permits drinking without raising the head while lying down. With a finger on the vent hole, the user can control the flow of liquid into the mouth without dribbling or spilling. The diameter of the mouthpiece opening is large enough to hold a straw. Includes 4, 6 and 8 oz. graduations. Steam autoclavable and top-shelf dishwasher safe.

To Use:

1. Remove the cap, fill with liquid and replace the cap.
2. Place the mouthpiece into the mouth with finger over the vent hole.
3. To start liquid flowing, remove finger from the vent hole.
4. To stop the liquid from flowing, place finger back over the vent hole.

Maddak Inc.

6 INDUSTRIAL ROAD • PEQUANNOCK, NJ 07440-1993 USA
973-628-7600 • fax 973-305-0841 • email custservice@maddak.com • www.maddak.com