


I N S T R U C T I O N S



Hole-in-One Utensils

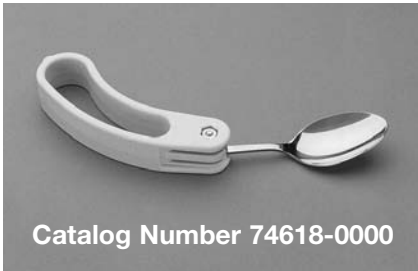

The handle can be held in two different ways, either with the fingers inserted through the hollow center or around the outside.

When not in use the utensil folds back toward the inner curve of the handle for compact storage.

Utensil comes set up for use by a **right-handed** person.

To switch to **left-handed** use:

1. Loosen the screw with a small screwdriver and pull the screw out of the holder.
2. Take the utensil out of the holder and turn it so that it is facing in the exact opposite direction. If the spacer in the second slot slips, realign it so that the hole in the spacer is aligned with the hole in the plastic.
3. Replace the screw and tighten.

This utensil can be tightened at any time by using a standard screwdriver.

Maddak Inc.

6 INDUSTRIAL ROAD • PEQUANNOCK, NJ 07440-1993 USA

973-628-7600 • fax 973-305-0841 • email custservice@maddak.com • www.maddak.com