

NEW!

Thermal Core Packs

Deep Penetrating Moist Heat Therapy



Pictured Left:
Half Pack #504 with
Terry Cover #853
(not included).

Pictured Below:
Standard #500 with
Foam Fill Cover #854
(not included).



- Moist heat relaxes sore muscles prior to treatment
- Multiple sizes available to treat all areas of the body
- Convenient, color-coded tabs provide visual indicator for pack rotation

Pictured Below:
Oversize #502 with
Foam Fill Cover #858
(not included).

Moist Heat Therapy

Core Products' new Thermal Core Packs use moist heat to relax sore muscles prior to professional treatment. The therapeutic moist heat radiating from these packs is more effective at providing deep penetrating heat to tissue than conventional heat packs.

Available in multiple sizes, Thermal Core Packs can be used on the back, neck and extremities to help relieve pain from:

- Arthritis
- Bursitis
- Backaches
- Sprains and Strains
- Muscle Soreness
- Headaches

Color Coded Tabs

Thermal Core Packs have two sets of color-coded tabs, which allow clinics to hang the packs in their heating units. The color-coded tabs also provide a visual indicator for pack rotation, ensuring that packs are fully heated prior to use.




"Making YOUR Life More Comfortable."

www.CoreProducts.com • 715.294.2050
808 Prospect Ave. • Osceola, WI 54020 USA

NEW!

Thermal Core Packs

Deep Penetrating Moist Heat Therapy

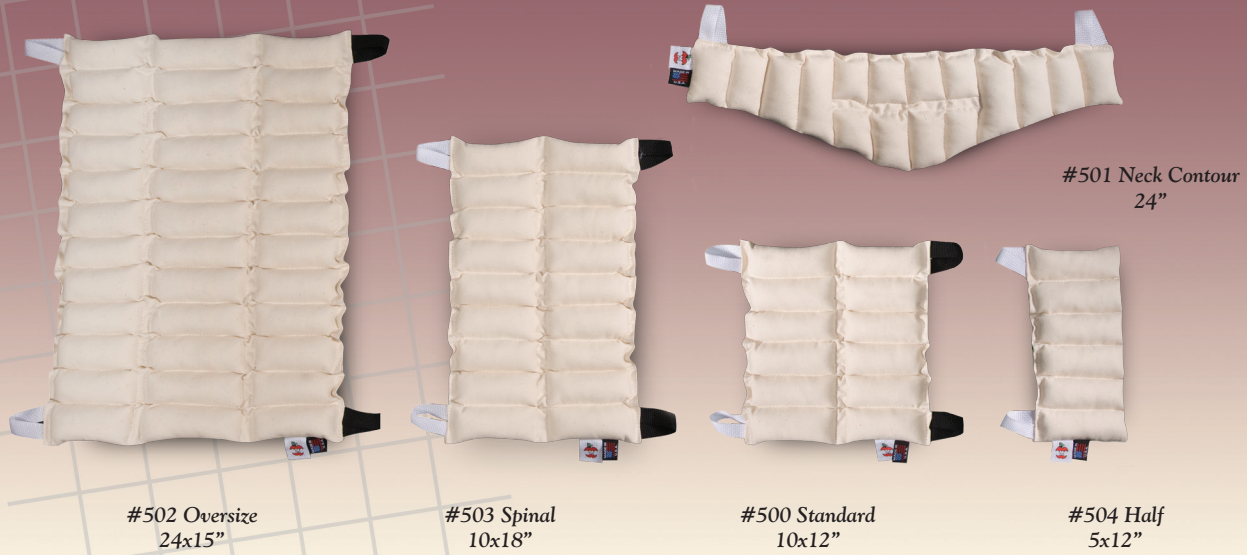

Instructions for Use

Thermal Core Packs should be heated in 160°F (71°C) water for at least 20 minutes prior to application to the desired area.

To ensure maximum effectiveness and patient safety, Thermal Core Packs should be placed in a protective cover before use. Thermal Core Pack Covers, also available from Core Products, are available in sizes to complement the Thermal Core Packs, allowing professionals to vary the heat intensity by adding or removing layers of insulation.

For moderate application of moist heat therapy, Core Products also offers foam lined Thermal Core Pack Covers.

Maintenance

Thermal Core Packs should remain in a heater or water bath after initial use, as the fill material in the packs may distort if previously conditioned packs are allowed to dry completely.

Cleaning your heating tank without chemicals is most beneficial to pack life. Tanks should be drained and wiped with a cloth and fresh water. No chemicals! Chemical agents leave residue that will cause premature disintegration of the pack.

For infrequent use, conditioned Thermal Core Packs can be stored in the freezer.



Pictured above:
Neck Contour #501 with
Foam Fill Cover #856
(not included).

Specifications

Thermal Core Packs

- #500 Standard 10x12"
- #501 Neck Contour 24"
- #502 Oversize 24x15"
- #503 Spinal 10x18"
- #504 Half Pack 5x12"

Thermal Core Covers

- #853 Standard Terry 19x27"
- #854 Standard Foam 19x27"
- #855 Neck Terry 25x16"
- #856 Neck Foam 25x16"
- #857 Oversize Terry 24x36"
- #858 Oversize Foam 24x36"

Color

- Cream; Black & White Tabs

Case Quantity

- Packaged per Case - 12

1
Year Warranty



Available From: