



Cardinal[®]

DETECTO

HS25KGP

Suspended Baby Scale

Owner's Manual



8535-M214-O1 Rev B
03/15

CARDINAL SCALE MFG. CO.
PO Box 151 • Webb City, MO 64870
Ph: 417-673-4631 • Fax: 417-673-5001
www.detectoscale.com

Printed in USA

Technical Support: Ph: 866-254-8261 • techsupport@cardet.com

UNPACKING

1. Open box.
2. Remove top Styrofoam insert from box.
3. Next, remove S-hook from insert and set aside.
4. Grasp scale and remove from box.
5. Remove Pants (baby sling seat) from box.
6. Remove bottom Styrofoam insert from box.
7. Turn insert over to locate second S-hook.
8. Remove S-hook from insert and set aside.
9. Remove scale from plastic bag.
10. Remove cardboard dowel between scale case and lower loop.
11. Scale is now ready to use.



Figure No. 1

OPERATION

1. Referring to Figure No. 1, install S-hooks on scale.
2. Unfold Pants (Baby Sling Seat).
3. Hang Pants on lower S-hook. See Figure No. 1.
4. Referring to Figure No. 2, turn Zero Adjust Knob until dial pointer is reading "0".
5. Place baby in Pants.
6. The dial pointer will indicate the baby's weight.
7. Remove baby from Pants and re-zero scale if necessary.



NEVER leave a baby unattended in the Baby Sling Seat. Failure to maintain control of the baby at all times while they are in the Baby Sling Seat can result in serious injury to baby.



Figure No. 2

CARE AND CLEANING

- **DO NOT** spray; water directly on the scale.
- **DO** clean scale using a damp, soft cloth and mild detergent.
NOTE! Some types of paper towels may scratch the dial lens.
- **DO NOT** use acetone, alcohol, or other volatile solvents for cleaning.

PARTS LIST

ITEM	QTY	PART NUMBER	DESCRIPTION
1	4	6600-1870	Lens Screw
2	1	6600-1871	Lens
3	2	6600-1872	S-Hook
4	1	HSPANT	Pants (Baby Sling Seat)

