


# Free2Go Rollator Assembly Instructions

## Parts of the Free2Go Rollator:

- ❶ Rollator frame
- ❷ Folding hinge with black metal cross bar
- ❸ Front wheels
- ❹ Metal screw, washer and knob (6 sets)
- ❺ Hand grip/brakes/rear wheel assembly
- ❻ Clips used to secure brake cable
- ❼ Padded/commode seat with pouch for wipes
- ❽ Backrest

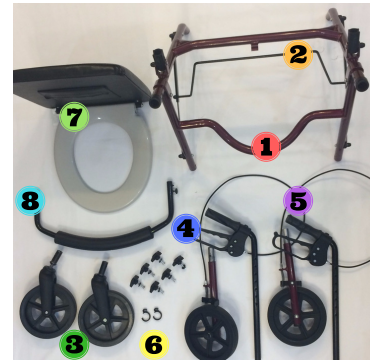


## Assemble: 4 Easy Steps (No tools required)

1. Remove rollator frame ❶ from box and lay on a flat surface. Secure black metal cross bar ❷ onto frame, allowing the folding hinge to lock in a straight position. Insert front wheels ❸ into openings at base of frame; secure with metal screw, washer and knob ❹



2. Insert hand grips with pre-assembled locking loop brakes ❺ into handlebar openings and adjust to desired height. Insert the metal screw into the selected hole and secure with washer and knob ❹. Insert wheels and secure with screw, washer and knob ❹. Secure brake cable to the rollator frame with the two (2) plastic clips ❻

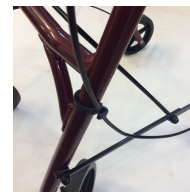

3. Center the padded/commode seat onto the frame; press down to secure onto rollator frame ❼



4. Insert foam back rest into openings on each side and secure in place ❽



## Fold:

Lift padded/commode seat and pull up on black metal cross bar, allowing unit to collapse.

**Act responsibly! Wipe commode seat top and bottom after use. A pouch to store disposable cleaning wipes is located on the under side of padded seat.**