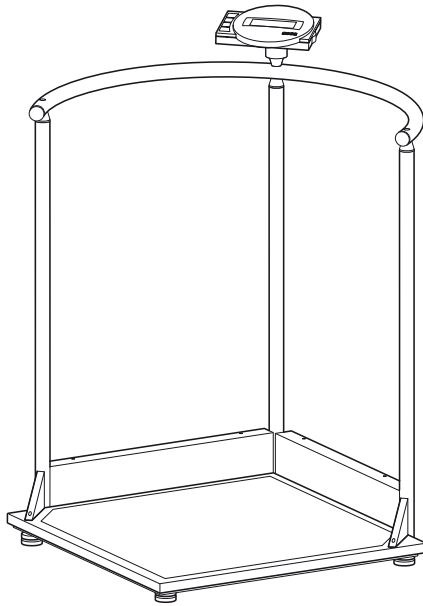
# seca 644

<b>D</b>	<i>Bedienungsanleitung und Garantieerklärung</i>	<i>2</i>
<b>GB</b>	<i>Instruction manual and guarantee</i>	<i>18</i>
<b>F</b>	<i>Mode d'emploi et garantie</i>	<i>34</i>
<b>I</b>	<i>Manuale di istruzioni e garanzia</i>	<i>50</i>
<b>E</b>	<i>Manual de instrucciones y garantía</i>	<i>66</i>
<b>P</b>	<i>Instruções de utilização e declaração de garantia</i>	<i>82</i>

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---

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# 1. Congratulations!

---

In the **seca 644** scale with standing aid, you have purchased a high-precision yet sturdy piece of equipment.

seca has been putting its experience at the service of health for over 150 years now, and as market leader in many countries of the world, is always setting new standards with its innovative developments for weighing and measuring.

In accordance with national regulations, the **seca 644** scale with standing aid is used primarily in hospitals, doctors' surgeries and in-patient nursing facilities.

The weight display can be switched between kilogrammes (kg), pounds (lbs) and stones (sts). Weight is determined within a few seconds.

In addition to the conventional determination of weight, the **seca 644** also has a

function for determining body mass index. For this purpose, height is entered on the keypad and the body mass index associated with the weight value is output.

You can use the pre-Tare function to record the weights of, e.g. three types of chair. This weight is automatically deducted in the weighing operation.

The large, secure standing aid also enables persons who are unsteady on their feet to stand safely on the scale.

The scale is of a very sturdy construction and will give you long and loyal service. It is easy to operate and the large display is easy to read.

The **seca 644** can be moved on castors and its low power consumption combined with its rechargeable batteries mean it will provide long, mobile service.

## 2. Safety

---

Before using the new scale, please take a little time to read the following safety instructions.

- Follow the instructions in the instructions for use.
- Keep the operating manual and the declaration of conformity contained therein in a safe place.
- Take measures to ensure that persons who have difficulty walking do not fall over when using the scale.
- The scale must be used with care. The upper surface may be slippery if wet!
- After moving the scale, please lower the scale slowly to the operating height.
- Do not drop the scale or subject it to violent shocks.
- Be sure to use only the seca mains unit supplied. Before plugging the mains unit into a socket, check that the mains voltage indicated on the mains unit matches local mains voltage.
- When using the scale with a mains unit, ensure that the supply cable is routed in such a way as to exclude any type of tripping hazard.
- Have repairs carried out only by authorised persons.

# 3. Before you start...

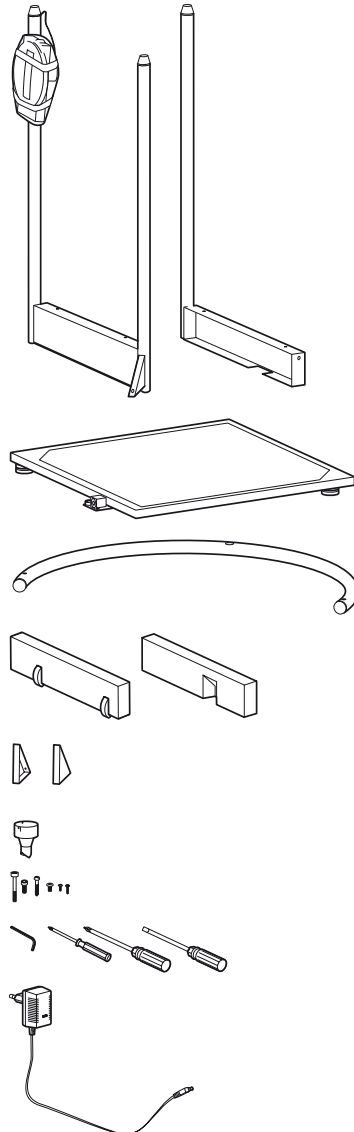
---

## Unpacking

- Remove the packaging, although leave the display head attached to the column. Place the scale platform on a solid, flat surface.

The scope of supply includes:

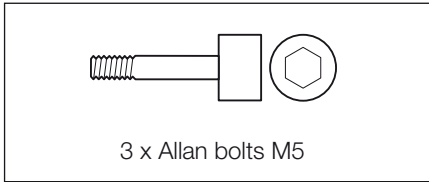
- Side section with two columns and the display head attached for transport
- Side section with one column
- Scale platform
- Railing
- Cover for the wheel casing
- Cover for the side casing
- 2 triangular covers right/left
- Display mount
- Set of assembly bolts
- Allan key, size 6
- Phillips screwdriver size 1 and size 3
- 1 screwdriver
- 1 seca mains unit
- 1 set of operating instructions

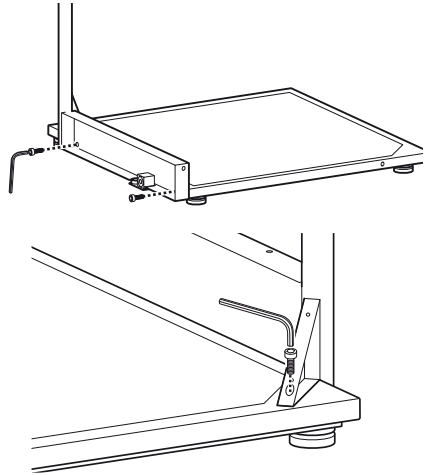
## Assembling the scale

### Step 1: Assemble the first side section

You will need the following bolts:

- Position the side section on the right-hand side of the scale platform.
- Bolt the side section to the scale platform using two Allan bolts.
- Use an Allan bolt to attach the triangular reinforcement to the scale platform.

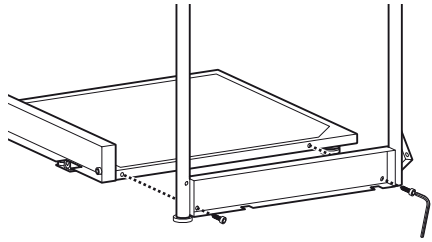
### Step 2: Assemble the side section with the wheel casing

You will need the following bolts:

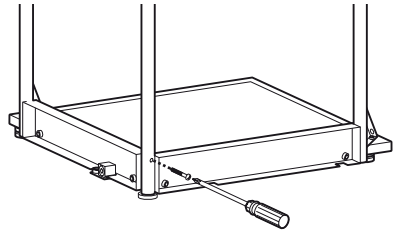
- Position the side section with the wheel casing on the rear side of the scale platform.
- Bolt the side section to the scale platform using two Allan bolts.

- Use an Allan bolt to attach the triangular reinforcement to the scale platform.

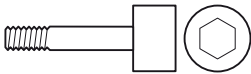
- Use the sheet metal bolt to attach the two side sections to the central column.

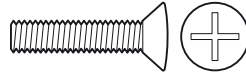
### Step 3: Assemble the railing

You will need the following bolts:

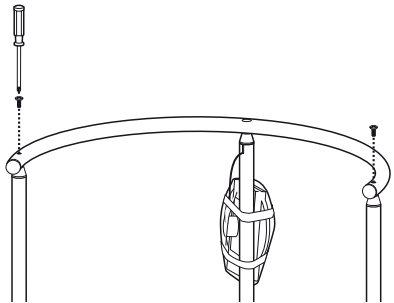
1 x Allan bolts M6

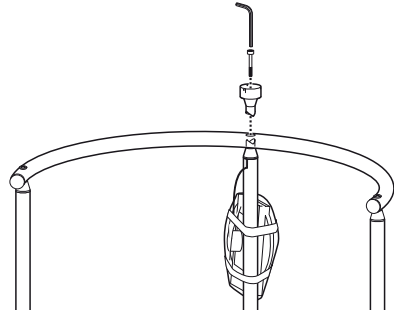
2 x Oval head bolt M6

- Position the railing on the three columns. The large bore holes must point downwards, and must be inserted into the heads of the columns.
- Use an oval head bolt to attach the railing to the two outer columns.

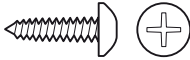
- Place the display mount with the peg-shaped end over the central column on the railing.
- Turn the display mount until the peg clicks into place.
- Screw the display mount tightly into place using the Allan bolt.

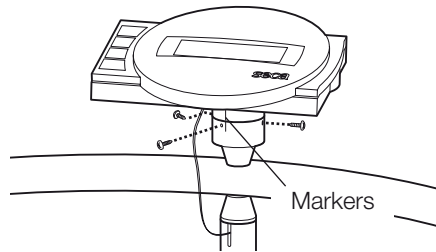
#### Step 4: Assemble the display head

You will need the following bolts:

3 x special Allan bolt,  $\varnothing$  2,5 mm

- Remove the transport protection from the display head.
- Position the display head over the display mount in such a way that the position markers are aligned.
- Use the three Allan bolts to attach the display head to the display mount.

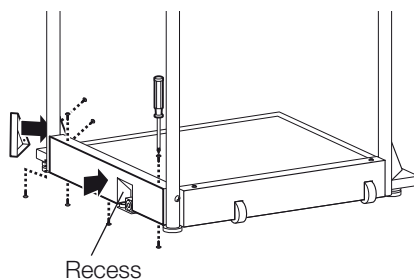
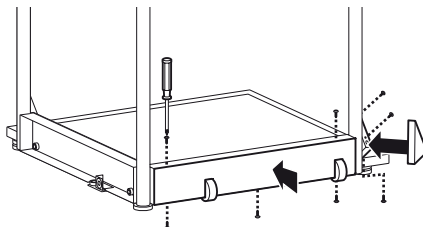
## Step 5: Assemble the side section covers

You will need the following bolts:

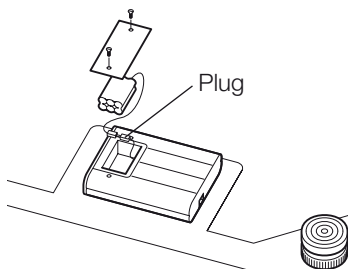
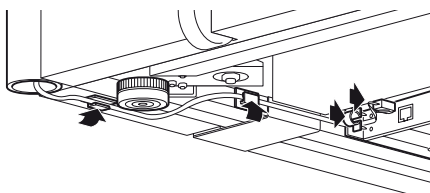
16 x Phillips bolts,  $\varnothing$  3,5 mm

- Place the scale on its side, with the rear side section facing upwards.
- Position the cover with the wheels over the side section, and attach this securely using 5 Phillips bolts: 3 x from below, and 2 x from above.
- Position the cover on the reinforcing triangle, and bolt this firmly in place using two bolts.
- Now turn the scale over so that the other side faces upwards.
- Position the cover with the recess pointing inwards over the side section, and attach this securely using 5 Phillips bolts: 3 x from below, and 2 x from above.
- Position the cover on the reinforcing triangle, and bolt this firmly in place using 3 bolts.

## Step 6: Assemble the electrical connections

- Connect the cable which comes out of the central column to the electronic box socket. In order to prevent measurement errors, lay the cable as shown in the diagram.
- Unscrew the bolts which secure the lid of the battery compartment, and remove.
- Connect the plug with the rechargeable battery block located in the battery compartment.
- Replace the lid of the battery compartment, and bolt this firmly in place.
- Turn the scale to the upright position.

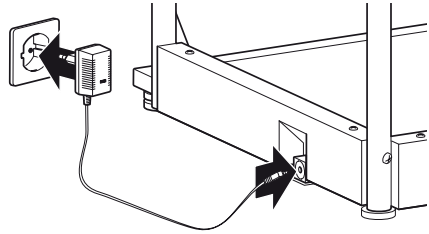
## Power supply

The scale is powered by a mains unit and by rechargeable batteries. The scale works independently of the mains, which is very handy for mobile use.

The LC display and the evaluation electronics have a low power consumption – 2,000 weighing operations can be performed with a fully charged battery. Once battery voltage is no longer adequate, the battery symbol  first appears in the display to indicate that battery power is dropping. If the batteries are not charged, battery voltage will ultimately not be adequate for error-free measuring. In this case, **bAtte** appears in the display and weighing is no longer possible.

The rechargeable batteries are charged by the mains unit supplied. For this purpose, the mains unit needs to be connected to the mains for at least 24 hours.

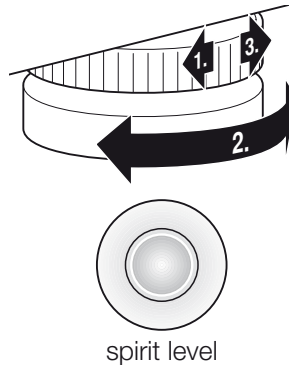
**Note: Please use only the mains unit supplied or mains units from the seca range of accessories. Before plugging the unit into the socket, check whether the mains voltage marked on the mains unit matches local mains voltage.**

## 4. Setting up and aligning the scale

- Place the scale on a level surface.
- Undo the locknuts using the four foot screws (1).

- Align the scale by undoing the foot screws. The air bubble in the spirit level must be right in the centre of the circle (2).
- Then re-tighten the locknuts (3).
- Ensure that only the feet of the scale are in contact with the floor. The scale may not be in contact at any other point.

### Important!

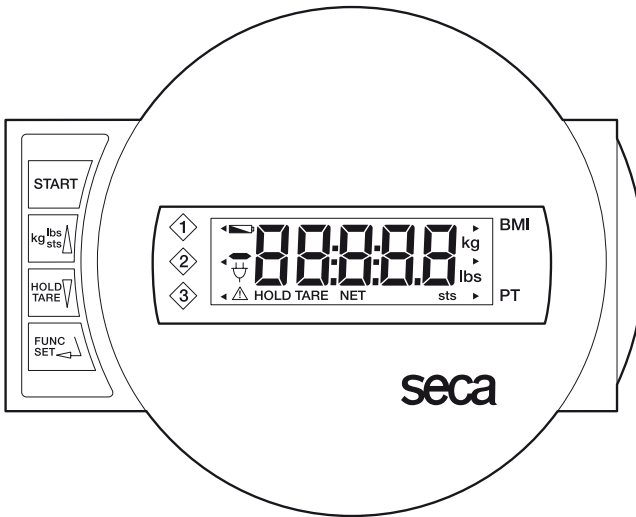
**The alignment of the base of the scale must be checked and corrected if necessary every time the location of the scale is changed.**

# 5. Operation

## Controls and displays

The controls and the display are on the display head on the column.

Switch scale on and off

press briefly → switch between kg and lbs  
hold down → switch between kg and sts  
in function menu → increase value, scroll

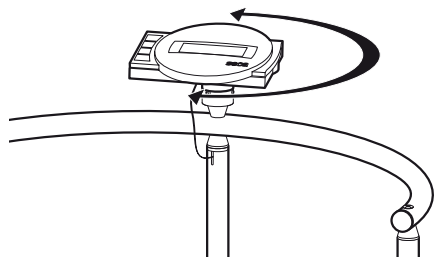
press briefly → activate/deactivate hold function  
hold down → set display to zero, activate/deactivate tare function  
in function menu → decrease value, scroll

Activate BMI or pre-Tare function, and confirm key for entries

In order to ensure that the display can be read from different positions, the display head can be rotated 180°.

## Correct weighing

- Press the Start key with no load on the scale.

SECA, 88888 and 0.0 appear consecutively in the display. The scale is then automatically set to zero and ready to use.

- If necessary, switch the weight display (see page 26).
- Ask the patient to mount the scale. Once on the scale, the patient should stand still.

### Note

The maximum load on each individual corner of the weighing platform is 200 kg.

- Read the weighing result off the digital display.
- If a load greater than 300 kg or 660 lbs is placed on the scale, **StOP** appears in the display.
- To switch off the scale, press the Start key again.

## Switching the weight display

The weight display can be switched between kilogrammes (kg) and pounds (lbs) or between kilogrammes (kg) and stones (sts).

- Switch on the scale using the green Start key. Weight is initially displayed in the last setting selected.
- To switch the weight display between kilogrammes and pounds, press the **kg/lbs/sts** key.
- The **kg** or **lbs** display lights up.
- To switch weight display between pounds and stones, hold down the **kg/lbs/sts** key for approx. 2 seconds. The **lbs** or **sts** display lights up.

## Automatic switch-off/economy mode

The scale automatically detects whether it is being operated on rechargeable batteries alone or whether it is connected to the mains.

- With rechargeable battery operation: automatic shut-down after 5 minutes.
- With mains operation: shut-down only via keypad.

## Taring weight display (TARA)

The Tare function is very useful if an additional weight, e.g. that of a chair, is to be ignored for weighing purposes.

Proceed as follows:

- Press the Start key with no load on the scale.

- First place the additional weight (e.g. the chair) on the scale and activate the Tare function by pressing and holding the **HOLD/TARE** key.

The display is reset to zero. **0.0** appears in the display. The note "NET" appears.

- Now place the patient on the scale. The patient may remain seated.

The scale determines the patient's weight. You can now perform as many weighing operations as you like.

- Hold the key down again to deactivate the Tare function; the scale is then back in normal weighing mode and the tare value has been erased.
- The tare value is likewise erased by the scale being switched off.

## Store weight value (HOLD)

You can also continue displaying the weight determined once the load has been removed. This makes it possible to look after a patient first, before having to note the weight.

- Press the **HOLD/TARE** key whilst the load is on the scale.

"HOLD" appears in the digital display. The value is stored until the scale switches off automatically.

- The HOLD function can be switched off by pressing the **HOLD** key again.

**0.0** appears in the display if the scale has no load on it, otherwise the current weight. The "HOLD" display vanishes.

### Note:

**The HOLD function can also be used if the TARE function has been executed beforehand.**

## Body-Mass-Index (BMI)

Body Mass Index puts height and weight in relationship to one another, providing more accurate information than ideal Broca weight, for example. A tolerance range is given which is considered ideal for health.

Three body sizes can be stored.

- Press the Start key with no load on the scale.

**SECA**, **88888** and **0.0** appear consecutively in the display. The scale is then automatically set to zero and ready for operation.

- Press the **FUNC** key.
- The last function used appears in the display.
- Use the arrow keys to select the BMI function **FU 1**. Activate the function by pressing the **FUNC** key once again.

- An arrow on the left-hand side of the display indicates which memory is set.

- Use the arrow keys to select the memory position under which you wish to record – or under which you have already recorded – the patient's height.

- Press the **FUNC** key again. The patient's recorded height will then appear on the display.

- You can change the value in 1 cm increments using the arrow keys.

- Once the correct value is set, press the **FUNC** key again. The BMI function is now activated.

- Ask the patient to mount the scale. Once on the scale, the patient should stand still.

- Read off the BMI on the digital display and compare it with the categories on page 29.

- Press an arrow key if you would like to see the weight briefly.

- To return to normal weighing mode, press the **FUNC** key.

- To switch off the scale, press the Start key again.

### Evaluating Body Mass Index for adults:

Compare the value determined with the categories quoted below which correspond to those used by WHO, 2000.

- **BMI below 18.5:**

Warning, this patient weighs too little. There could be a tendency to anorexia. An increase in weight is recommended to improve well-being and performance. If in doubt, consult a specialist.

- **BMI between 18.5 and 24.9:**

The patient is of a normal weight. He can stay as he is.

- **BMI between 25 and 30 (pre-obese):**

The patient is slightly to moderately overweight. He should cut his weight if he is already ill (e.g. diabetes, hypertension, gout, fat metabolism disorders).

- **BMI over 30:**

Weight reduction is essential. Metabolism, cardiovascular system and bones are all under strain. A consistent diet, plenty of movement and behavioural training are all recommended. If in doubt, consult a specialist.

### Weighing with pre-Tare

If the pre-Tare function is activated, a stored weight value is deducted from the currently measured weight. This function can be useful, for example, when the patient and the carer mount the scale together, and the weight of the carer is not meant to be shown.

Three different tare weights (carer weights) can be stored.

- Press the Start key with no load on the scale.

**SECA**, **88888** and **0.0** appear consecutively in the display. After that, the scale is automatically set to zero and ready for operation.

- Press the **FUNC** key.

The last function used appears in the display.

- Use the arrow keys to select the pre-Tare function **FU 3**. Activate the function by pressing the **FUNC** key once again.

An arrow on the left-hand side of the display indicates which memory is set.

- Use the arrow keys to select the desired memory for storing the additional weight or where a weight may already be stored.

- Then press **FUNC**. In the display you will see the tare value from the memory.
- You can alter the weight of the patient using the arrow keys in display steps.
- Once the correct value is set, press the **FUNC** key again.  
The pre-Tare function is now activated.
- Mount the scale together with the patient. Read the weighing result off the digital display.
- Press an arrow key if you would like to see the current weight value briefly.
- To return to normal weighing mode, press the **FUNC** key.
- To switch off the scale, press the Start key again.

## Setting damping

Damping is useful for reducing weight determination malfunctions (due to patient movements, for example). The stronger the damping, the more slowly the weight display reacts. You can set damping to suit requirements.

- Press the **FUNC** key and keep it depressed for 2 seconds.

**FIL** and the current setting appear in the display:

- 0 = slight damping  
→ fast determination of weight
- 1 = average damping  
→ normal determination of weight
- 2 = strong damping  
→ slow determination of weight

- Press an arrow key if you want to change the set value.

- Press the **FUNC** key again to store the displayed value. The scale is then back in normal weighing mode.

**The settings are retained even once the scale is switched off.**

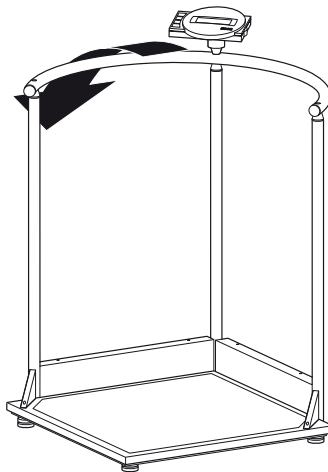
## 6. Moving the scale

---

For mobile deployment in a ward, the scale can be tipped backwards and moved about on the rollers.

### **Important!**

**The alignment of the base of the scale must be checked and corrected if necessary every time the location of the scale is changed.**

## 7. Cleaning

---

Clean the rubber surface and the housing as required using a domestic cleaning agent or commercially-available disinfectant. Follow the manufacturer's instructions.

Under no circumstances use abrasive or acid cleaners, white spirit, benzene or the like for cleaning. Such substances can damage the high-quality surfaces.

## 8. What do I do if...

---

### **... no weight display comes on?**

- Is the scale switched on?
- Check the mains connection and the connection of the battery pack.
- Is the plug contact for connecting the display to the scale correctly connected?
- Switch the scale off for a few seconds. This can be done by opening the plug connected to the rechargeable battery block and unplugging the mains supply from the scale.

### **... does not appear before the weighing operation?**

- Press the Start key again – there must not be any load on the scale – and only its feet should be in contact with the floor.

### **... one segment is illuminated constantly or not at all?**

- The relevant segment has a fault. Inform Servicing.

### **... the display shows ?**

- Battery voltage is dropping. Charge the rechargeable batteries in the next few days.

### ... **bAŁŁ** appears in the display?

- The rechargeable battery is so severely discharged that weighing is no longer possible. Recharge the battery.

### ... **5ŁOP** appears in the display?

- Maximum load has been exceeded.

### ... **the display flashes?**

- If you have just activated the Hold or Tare function, please wait until the scale has detected a steady weight value. The display will then stop flashing.
- If there has been no activation of a function beforehand, remove the load from the scale and wait until **00** is displayed, then weigh again.

### ... **the display shows ŁENP?**

- The ambient temperature of the scale is too high or too low. Place the scale in an ambient temperature between +10 °C and +40 °C. Wait about 15 minutes for the scale to adapt to the ambient temperature and then weigh again.

### ... **the display E and a number appear?**

- Switch off the scale with the Start key and start the scale again. The scale will then work normally again.  
If this is not the case, interrupt the power supply by briefly removing the rechargeable battery. If this measure is equally unsuccessful, inform the Service department.

## 9. Maintenance

---

Your seca scale leaves the factory with an accuracy greater than  $\pm 0,15\%$ . To ensure that this accuracy is maintained, the product must be carefully set up and regularly maintained. Depending on the frequency of use, we recommend servicing every 3 to 5 years. Please ensure that a qualified service agent is used. If in doubt, contact seca Customer Service for further information.

## 10. Technical data

---

<b>Dimensions: (width, length, height)</b>	750 x 750 x 1140 mm / 30 x 30 x 45 ins
<b>Size of platform</b>	600 x 600 mm / 23.6 x 23.6 inch
<b>Height of platform</b>	45 mm / 1.77 inch
<b>Weight</b>	approx. 24 kg / 53 lbs
<b>Temperature range</b>	+10 °C to +40 °C
<b>EN 60 601-1:</b>	
insulated equipment, Protection Class II	
medical electrical equipment type B	
<b>Accuracy</b>	greater than $\pm 0,15\%$
<b>Colour</b>	black/white
<b>Maximum load</b>	300 kg / 660 lbs / 47 sts

**Graduations**  
**Power supply:**

100 g / 0.1 lbs  
model dependent mains unit  
and built-in rechargeable battery pack

## 11. Spare parts and accessories

---

**Rechargeable batteries**

seca no.: 68-22-12-721

**Model dependent mains unit**

Plug-in mains unit, Euro: 230V~ / 50Hz / 12V= / 130mA

seca no.: 68-32-10-252

Switchmode power adapter, 100-240V~ / 50-60Hz / 12V= / 0.5A

seca no. 68-32-10-265

## 12. Disposal

---

### Disposing of the device

Do not dispose of the device in domestic waste. The device must be disposed of properly as electronic waste. Follow the national regulations which apply in your case. For further information, contact our service department at: [service@seca.com](mailto:service@seca.com)

### Batteries or rechargeable batteries

If you use batteries for your scale, do not dispose of them in domestic waste. Dispose of used batteries at local collection points. When buying new batteries, select those low in harmful substances and containing no mercury (Hg), cadmium (Cd) or lead (Pb).

## 13. Warranty

---

A two-year warranty from date of delivery applies to defects attributable to poor materials or workmanship. All moveable parts - batteries, cables, mains units, rechargeable batteries etc. - are excluded. Defects which come under warranty will be made good for the customer at no charge on production of the receipt. No further claims can be entertained. The costs of transport in both directions will be borne by the customer should the equipment be located anywhere other than the customer's premises. In the event of transport damage, claims under warranty can be

honoured only if the complete original packaging was used for any transport and the scale secured and attached in that packaging just as it was when originally packed. All the packaging should therefore be retained.

A claim under warranty will not be honoured if the equipment is opened by persons not expressly authorised by seca to do so.

We would ask our customers abroad to contact their local sales agent in the event of a warranty matter.

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