


# VivaGuard<sup>®</sup>

Lancing Devices, Lancets and Safety Lancets

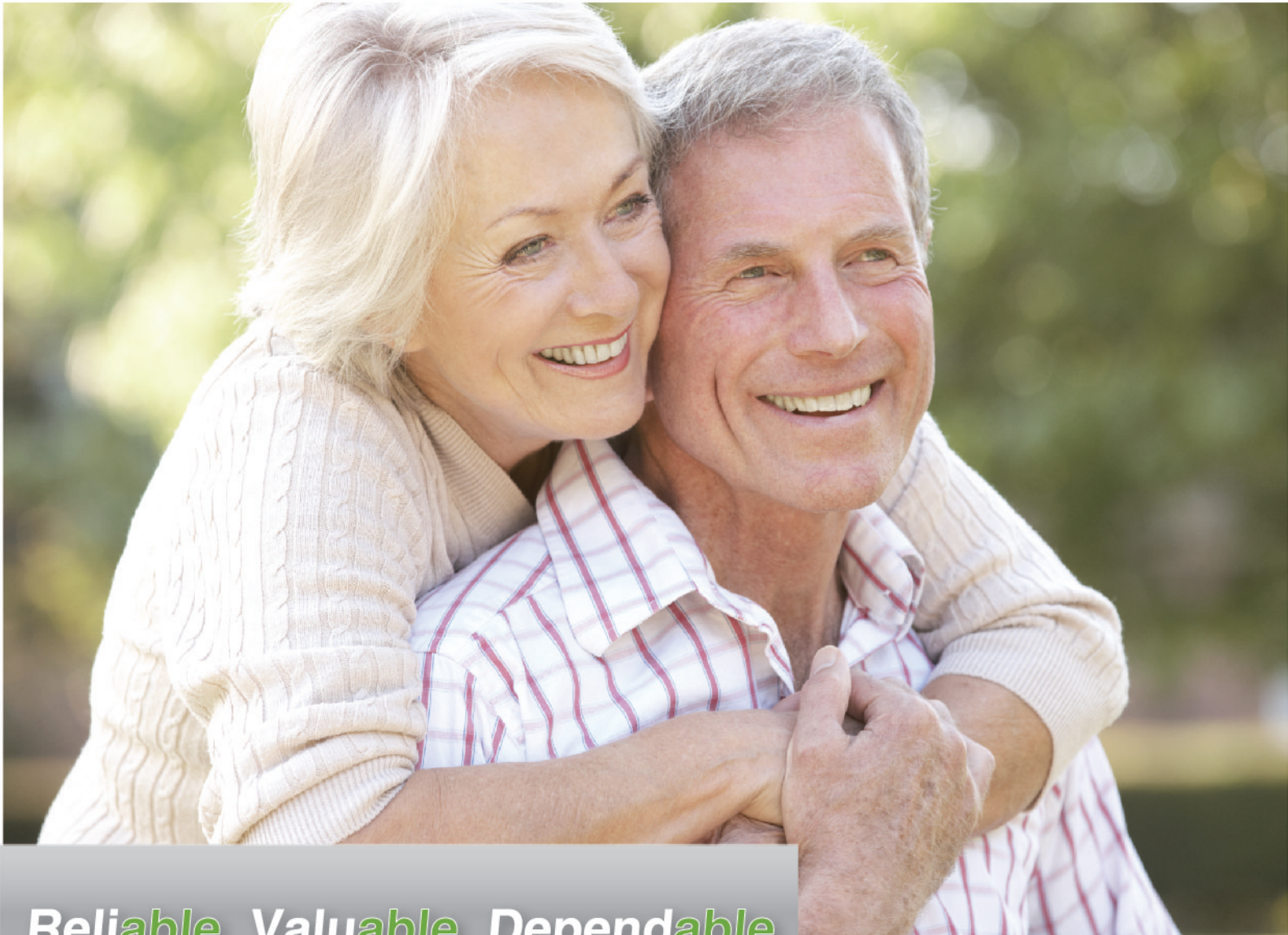

**Reliable Valuable Dependable**

**Able** Diagnostics, Inc.

## VivaGuard® Lancing Device

- Lancing Device with 5 depth settings offers maximum comfort through selecting a proper penetration depth
- Easy to carry with a secure-grip body design for safety
- Smooth gliding mechanism offers reliability and simplicity
- Unique Design and Easy to Use - Featuring a **Lancet Ejector** that easily disposes of the used lancet, hands-free, to prevent accidents and for better safety
- Convenient storage with 5-year shelf life



Lancet Ejector

## VivaGuard® Lancets

- 30G Lancets with extra sharpened needlepoints to minimize the discomfort of finger pricking
- Designed to fit most commonly used Lancing Devices
- Additional gauges are available upon request




## VivaGuard® Safety Lancets


- Ergonomically designed Safety Lancets for individual use
- Easy to carry with anti-skid body design to ensure safe grip
- Easy to use - just twist and pull off the protective cap, place it on the puncture site and press to collect blood sample
- 28G Sterile Safety Lancets with extra sharpened needlepoints to minimize discomfort
- Defined penetration depth at 1.8 mm for optimal sampling
- Well controlled small penetration area for precision
- Needle automatically retracts and locks, to eliminate re-use and cross-infection for ensured safety
- Convenient storage with 5-year shelf life
- Additional gauges are available upon request




### Steps of use safety lancets

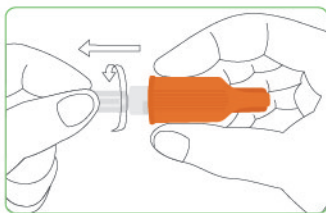

- ▶ Twist off the protective cap and then pull it out.

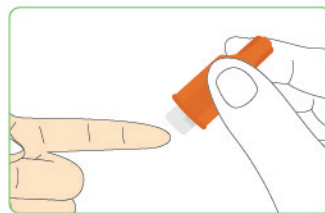

- ▶ Place the lancet against the puncture site.

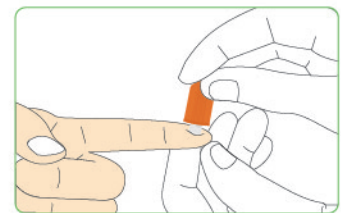

- ▶ Gently push the lancet down against the puncture site.

Item	Gauges	Catalog No.	Package (kit/CTN)
VivaGuard® Lancing Device	----	VGD01-381	400
VivaGuard® Lancet 100s	30G	VGL01-383	200
VivaGuard® Safety Lancets 100s	28G	VGL02-382	50



**Able Diagnostics, Inc.**

12396 World Trade Drive, Suite 216, San Diego, CA 92128, USA

Tel: 1 (858) 309-8565

Email: [info@ablediagnostics.com](mailto:info@ablediagnostics.com)

Website: [www.ablediagnostics.com](http://www.ablediagnostics.com)